
# SR Ivocap® Elastomer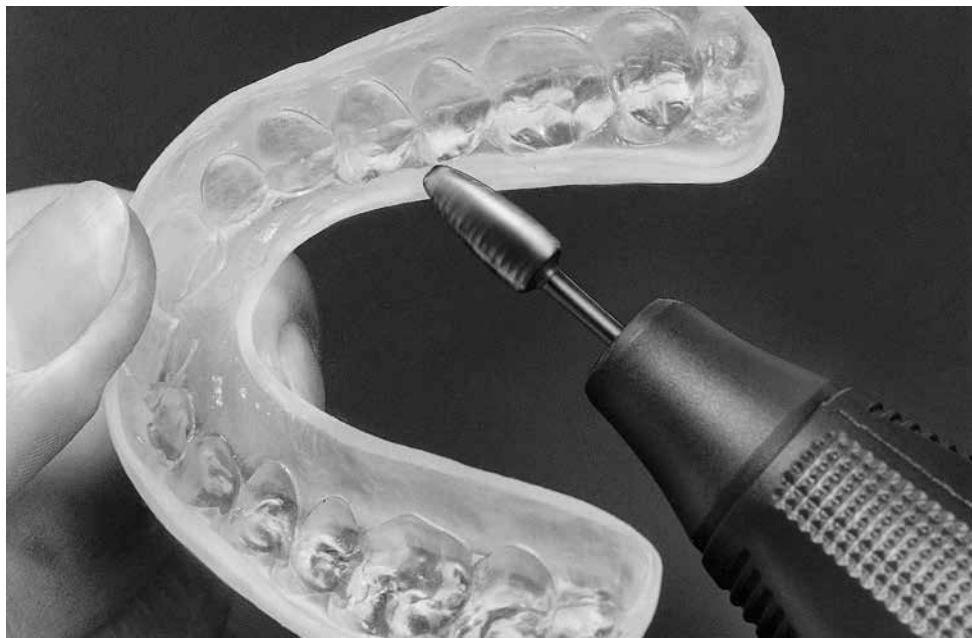

**EN Instructions for Use**

Page 2

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Pagina 14

**DE Gebrauchsinformation**

Seite 6

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Página 18

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Page 10

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Página 22

Rx ONLY

Date information prepared:

563716/2016-10-16/WE1/Rev.1

**Manufacturer**

Ivoclar Vivadent AG

Bendererstrasse 2

9494 Schaan/Liechtenstein

[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**ivoclar**  
**vivadent**<sup>®</sup>  
*technical*

# Instructions for Use

SR Ivocap Elastomer is a specially developed acrylic resin with an elastomer base. Due to its physical properties and elasticity, the material, which is processed using the SR Ivocap System in the laboratory, is suitable for a variety of indications.

## Properties

- **accurate fit**
- transparent
- high tensile strength
- high tear resistance

## Indication

- positioner
- mouthguards for various sports, eg boxing, ice hockey, judo, etc
- bruxism splints
- bite splints requiring elasticity

## Contraindication

- Unpolymerized material should not come into contact with intraoral areas
- The product should not be used if the patient is known to be allergic to any of the components
- Amalgam fillings may discolour mouthguards or bruxism splints made of the Ivocap Elastomer. In such cases, the product should no longer be used

## Advantages

- shrinkage compensation due to the SR Ivocap processing technique
- comfortable for patients
- odourless, tasteless

## Composition

	wt%
<b>Powder:</b>	
Styrene polyolefin copolymer	96.8
Citrate softener	1.4
Catalyst	1.8

<b>Liquid:</b>	
Methyl methacrylate	100.0

## Contents of capsule:

20 g polymer  
17.5 ml monomer

## Delivery form

10 capsules powder, 20 g each  
10 flasks liquid, 17.5 ml  
10 funnels


### Important processing instructions

- Always prepare **two joined injection channels** for polymerization
- Possible dough residue dried on the inner wall of the capsule does not affect the quality of the polymerized work
- Pre-inject for **10 minutes** before transferring the flask to the water bath
- The polymerization time from boiling point is **45 minutes** under 6 bar pressure
- Two capsules of mixed SR Ivocap Elastomer are required to fabricate a positioner
- The compositions of the monomers of SR Ivocap Elastomer, SR Ivocap High Impact and SR Ivocap Clear are not identical. Make sure to prevent mix-ups of these three monomers, as confusing them may lead to product errors.

Fully model mouthguard on the model (wax min 2 mm thick).

### Modelling of a mouthguard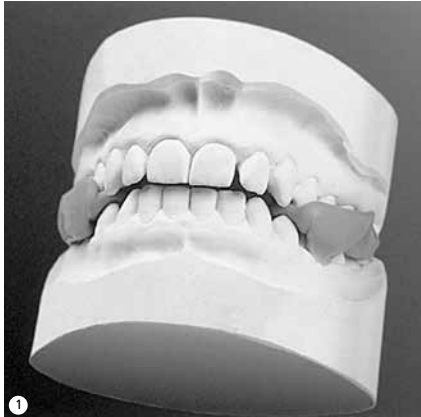

Mount plaster working model in the articulator with the help of working bites.

Invest model in the SR Ivocap flask, always preparing two injection channels. Before investing a positioner, shorten both models at the base.

## Polymerization with the SR Ivocap Technique

Preheat the flask in hot water for 5 minutes, then open immediately and thoroughly boil out the wax. Insulate the still warm **flask twice with Separating Fluid**. Then, put the flask in the clamping frame and apply pressure. Pour the monomer into the polymer capsule. Push the plunger far enough into the capsule for the empty monomer container to be re-inserted into the base. Close the capsule with the plug.


Mixing time in Cap Vibrator: **5 minutes**

Should there be **dried dough residue** on the inner wall of the capsule, it will not affect the quality of the polymerized work.

Use the capsule plunger to force excess air out of the capsule.

Now, inject the prepared capsule's contents into the flask.

Two capsules of mixed SR Ivocap Elastomer are required for fabricating a positioner.

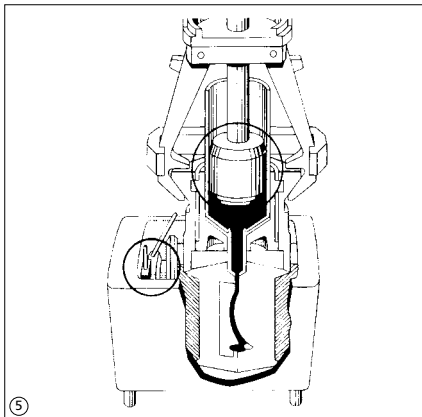Leave the flask under 6 bar pressure for **10 minutes** before placing it, still under pressure, in a hot water bath to enable the elastomer dough to flow in thoroughly. Once the boiling point is reached, polymerization takes **45 minutes** in boiling water. Subsequently, place the pressurized flask in cold water for at least 20 minutes.

## Finishing

After removing the investment, detach the injection channels and any large pieces of excess material.

Finish with 12-16 bladed resin burs. Then, finish with emery paper (rough and fine).

Correct final details with rubber polishers. Prepolish with wet pumice and a soft brush, using a bench motor.

Polish with a clean, dry cloth wheel, using a bench motor.

### Do not use polishing paste containing ammonia or solvents!

Remove SR Ivocap Elastomer material using ultrasonic equipment or clean with a brush and cleaning medium.

### Safety notes

- The monomer contains methyl methacrylate (MMA)
- MMA is highly inflammable and irritating – flash point 10 °C / 50 °F
- MMA may irritate eyes, skin and respiratory organs
- Sensitization by skin contact is possible
- Avoid skin contact with monomer and uncured material. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates
- Store the monomer bottle in a cool, dark, well-ventilated place
- Do not inhale vapours
- Keep the material away from sources of heat – do not smoke
- Do not dispose of the monomer in the drainage system
- Avoid electrostatic charging

### Storage

- Do not use the material after the date of expiration
- Storage temperature: 2–28 °C / 36–82 °F
- Store mixed material at 8 °C / 46 °F in a cool, dark place for max 5 days
- If the material is refrigerated, remove the required quantity of stored material one hour prior to using it to enable it to reach room temperature
- Store polymerized SR Ivocap Elastomer splints or positioners in places protected from light
- Keep out of the reach of children

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**