Indications Recommended for inlays, onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, posts, short and long span bridges, PFM crowns.

Composition Au Ni Cr Pd Ag Cu Zn 15 P 25 Pb 69 Ni 55 In 10 10 10 10

Instructions for Use
Modo d'empiego
Instrucciones de uso
Gerbruchsinformation
Instrucciones de uso
Briksanvisning
Modo d'empiego
Instrucciones de uso
Käyttöohjeet
Briksanvisning

White, gold-based dental metal casting alloy, Type 4

ivoclar Vivadent Worldwide

Australia ivoclar Vivadent Pty. Ltd. 1 - 5 Overseas Drive P.O. Box 367 Noble Park, Vic. 3174 Tel. +61 3 979 595 99 Fax +61 3 979 596 45 www.ivoclarvivadent.com.au	Brazil ivoclar Vivadent Ltda. Alameda Caiapós, 723 Centro Empresarial Tamboé CEP 06460-110 Barueri - SP Tel. +55 11 2424 7400 Fax +55 11 2424 7440 www.ivoclarvivadent.com.br	Canada ivoclar Vivadent Inc. 1-8600 Dixie Road Mississauga, Ontario L5T 2Y2 Tel. +1 905 670 8499 Fax +1 905 670 3102 www.ivoclarvivadent.us	China ivoclar Vivadent (Shanghai) Trading Co., Ltd. 2/F, Building 1, 881 Wuding Road Jing An District Shanghai 200040 Tel. +86 21 6032 1657 Fax +86 21 6176 0988 www.ivoclarvivadent.cn	Colombia ivoclar Vivadent Marketing Ltd. Calle 134 No. 7-8-83, Of. 520 Bogotá Tel. +57 1 627 33 99 Fax +57 1 627 33 99 www.ivoclarvivadent.co	France ivoclar Vivadent SAS B.P. 118 74410 Saint-Jorioz Tel. +33 450 88 64 00 Fax +33 450 68 91 52 www.ivoclarvivadent.fr	Germany ivoclar Vivadent GmbH Dr. Adolf-Schneider-Str. 2 73479 Ellwangen, Jagst Tel. +49 79 61 8 89 0 Fax +49 79 61 83 26 www.ivoclarvivadent.de	India ivoclar Vivadent Marketing (India) Pvt. Ltd. 503/504 Rajaja Plaza 15 B Shah Industrial Estate Veera Desai Road, Andheri (West) Mumbai, 400 053 Tel. +91 22 2673 0302 Fax +91 22 2673 0301 www.ivoclarvivadent.in	Italy ivoclar Vivadent S.r.l. Via Isozno 6769 40033 Casalecchio di Reno (BO) Tel. +39 051 6113555 Fax +39 051 6113565 www.ivoclarvivadent.it	Japan ivoclar Vivadent K.K. B.P. 118 1-28-24-4F Hongo Bunkyo-ku Tokyo 113-0033 Tel. +81 3 6903 3535 Fax +81 3 6904 3657 www.ivoclarvivadent.jp	Sweden ivoclar Vivadent AB Dalvägen 14 169 56 Solna Tel. +46 8 514 93 930 Fax +46 8 514 93 940 www.ivoclarvivadent.se	Turkey ivoclar Vivadent Liaison Office Tesvikiye Mahallesi Sakayik Sokak Nisantasi Plaza No: 40 Kat: 6 Daire: 31-32 34021 Sisi Istanbul Tel. +90 212 343 0802 Fax +90 212 343 0842 www.ivoclarvivadent.com.tr	UK ivoclar Vivadent Limited Compass Building Fildspar Close Warrens Business Park Enderby Leicester LE19 4SE Tel. +44 116 284 78 80 Fax +44 116 284 78 81 www.ivoclarvivadent.co.uk	Russian Federation ivoclar Vivadent Marketing Ltd. Prospekt Andropova 18, Korpus 6 Office: 10-06 115432 Moscow Tel. +7 499 418 0300 Fax +7 499 418 0310 www.ivoclarvivadent.ru	Spain ivoclar Vivadent S.L.U. Ctra. Fuencarral, 24- Portal I Bajó 28108 Alcobendas (Madrid) Tel. +34 91 75 78 20 Fax +34 91 75 78 38 www.ivoclarvivadent.es
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

INSTRUCTIONS FOR USE

MODELLATION
Design the framework in a reduced anatomical shape taking the planned veneer into consideration. Single crowns require a thickness of minimum 0.3 mm; abutment crowns thickness a minimum of 0.5 mm. Make sure the framework demonstrates adequate stability of shape. Avoid sharp angles. Design the connector areas to be adequate for the position and alloy being used.

SPRUIING
Provide the modelled bridge framework or coping with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center. The connection sprues between the reservoir and the coping should be 2.5-3.0 mm in length and width.

INVESTMENT
Weigh the wax pattern including the sprue to determine the quantity of the alloy to be used. (See wax conversion sheet/formula: weight x density = gr. of alloy). Use investment following the manufacturer's instruction.

BURN-OUT
The suggested burnout temperature: 750-820C/1380-1510F

MELTING AND CASTING
Use a separate carbon/ceramic crucible for each alloy and preheat the crucible (ceramic) in the burnout furnace. Used and new alloys must be in a ratio of 1:1. Depending on the type of casting machine, follow the manufacturer's instructions for use. If you use Ivoclar Vivadent Magic Wand, set propane to 0.35 bar/5 psi and oxygen to 0.7 bar/10 psi pressure. Keep the alloy in the reducing atmosphere of the flame between the inner and outer cones. Do not use flux. After casting bench cool to room temperature.

Casting Temperature: 1330-1390C/2435-2535F

METAL PREPARATION
Carefully divest and clean the object with Al₂O₃. Do not use a hammer for divesting the object to prevent deformation. Finish the framework with carbide burs or with ceramic-bonded grinding instruments. Avoid inhalation of dust during grinding!

OXIDATION
Blast surface with 50-100 micron Al₂O₃ at max. 4.5 bar/65 psi pressure before oxidation. Subsequently, clean with ultrasonic or steam. Place the object on the firing tray and provide adequate support. Place the tray in a porcelain flame set at a low temperature of 650C/1200F and increase the flame temperature to 950C/1740F without vacuum with 1 min. hold time at top temperature.

HEAT TREATMENT
Hardening: 600C/1110F for 20 minutes; air cool.

SOLDERS AND FLUXES
Design the soldering pattern as small as possible and preheat it in the furnace at approximately 600C/1112F. The soldering gap should be the same thickness as the soldering strip. Allow the object to cool slowly after soldering.

Pre-Solder: High Fusing White Ceramic Solder
Post-Solder: .615 Fine Solder, Low Fusing White Gold Solder, .585 Fine Solder, Bondal Flux
Laser weld material: Laser Ceramic White

POLISHING
After glazing remove oxide and flux residue and finish/polish the framework with rubber finishes/polishers.

INDICATIONS
Recommended for inlays, onlays, 3/4 crowns, crowns, telescope crowns, conus crowns, posts, short and long span bridges, PFM crowns.

CONTRAINDICATIONS
For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS
In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS
Galvanic effects may occur between different alloys in the same oral environment.
For additional information look into the alloy property chart.

SPRUIING METHOD

RECOMMENDATIONS
DIRECT: single crowns, inlays and onlays
INDIRECT: multiple units and multiple single crowns

INSTRUCTIONS:
1. Select a sprue with a reservoir equal to or larger than the thickest cross-section of the restoration.
2. Maintain the reservoir(s) within the heat center of the investment; keep the restoration(s) approximately 5 mm from the end of the investment and no closer than 5 mm from the sides.
3. Make sure that the sprue is connected to the thickest part of the restoration.
4. The connection between the sprue and the restoration should be flared ("trumpet" configuration); eliminating turbulence of the alloy (causing investment erosion) and facilitating an undisturbed flow of alloy during casting and solidification.
5. Maintain proper alloy weight in order to eliminate the negative effect of a too large button during solidification. The rule of thumb for proper alloy weight determination is: total wax weight x specific gravity of the alloy = proper alloy weight.

SUGGESTIONS:
1. Thickness and design of wax pattern: follow your manufacturer's instructions.
2. Use chill-vents when casting heavy and/or large restorations.

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