# Monobond N

English

## Instructions for Use

### Mode d'emploi

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### Kullanma Talimatı

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## Instructions for Use

### Description

Monobond N is a universal primer mediating an adhesive bond between luting composites (particularly those of the Variolink N- and Multilink N-product lines) and all materials used for indirect restorations (glass- and oxide ceramics, metal, composite resin, fibre-reinforced composite).

### Composition

Alcohol solution of silane methacrylate, phosphoric acid methacrylate and sulphide methacrylate.

### Indication

Monobond N serves as bonding agent and is used to create a durable chemical bond between luting composites and glass-/oxide ceramic, metal, composite and fibre-reinforced composite restorations.

### Contraindications

- Any use not listed in the indication
- If a patient is known to be allergic to any of the ingredients of Monobond N

### Side effects

Systemic side effects are not known to date. In rare cases, components of Monobond N may lead to sensitization. Monobond N should not be used in such cases.

### Interactions

Contamination with saliva, blood or water must absolutely be avoided during the entire procedure.

### Usage

#### Adhesive cementation of prosthetic restorations

- 1 Preparing the restoration
- 1.1 Glass-ceramic restorations (e.g. IPS Empress®)
  - Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 60 seconds or according to the

For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

645759/0512/6spr/G

ivoclar  
vivadent®  
clinical

- instructions of the manufacturer of the restorative material.
- Thoroughly rinse the restoration with water spray and dry with oil-free air.
- 1.2 Lithium disilicate glass-ceramic restorations (e.g. IPS e.max® Press, IPS e.max CAD)
- Etch with 5% hydrofluoric acid (e.g. IPS Ceramic Etching Gel) for 20 seconds or according to the instructions of the manufacturer of the restorative material.
  - Thoroughly rinse the restoration with water spray and dry with oil-free air.
- 1.3 Restorations made of zirconium oxide (e.g. IPS e.max ZirCAD) or aluminium oxide ceramics
- Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material).
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
  - Thoroughly rinse the restoration with water spray and dry with oil-free air.
- 1.4 Metal or metal-supported restorations
- Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer) until an even matt surface has been achieved.
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
  - Thoroughly rinse restoration with water spray and dry with oil-free air.
- 1.5 Composite or fibre-reinforced composite restorations
- Sandblast the restoration surface (according to the instructions of the manufacturer of the restorative material).
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
  - Thoroughly rinse the restoration with water spray and dry with oil-free air.
- 2 Apply a thin coat of Monobond N with a brush or a Microbrush to the pre-treated surfaces. Allow the material to react for 60 seconds. Subsequently, disperse any remaining excess with a strong stream of air.

- 3 Apply the luting composite (e.g. Variolink N or Multilink N) according to the respective Instructions for Use and seat the restoration.

#### Additional notes

- Contamination with saliva, blood or water must absolutely be avoided during the entire procedure. In case of contamination, the entire treatment of the restoration is to be repeated (see Section 1).
- If the primer dries up on the surface of the restoration before blasting with a strong stream of air, the surface needs to be rinsed with pure alcohol and blown with air.
- **Important:** In order to create a strong bond, do not clean the zirconium oxide or metal surfaces with phosphoric acid. Instead, the use of Ivoclean universal cleaning paste is recommended.
- Do not insert the brush into the Monobond N bottle. This may lead to contamination.
- Carefully close the Monobond N bottle immediately after use.

#### Warning

- The product contains ethanol. Keep away from sources of ignition.
- Monobond N is an irritant. Avoid contact with the skin and mucous membrane. If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

#### Shelf life and storage

- Do not use Monobond N after the indicated date of expiration.
- Storage temperature 2–28 °C(36–82 °F)
- Shelf life: see expiry date on label.

**Keep out of the reach of children.  
For use in dentistry only.**

## Date information prepared

05/2012, Rev. 0

## Manufacturer

Ivoclar Vivadent AG

FL-9494 Schaan / Liechtenstein

This material has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

The Material Safety Data Sheet is available online at

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## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**