Description

Monobond Etch & Prime is used to condition silica-based ceramic surfaces for the adhesive bond with luting composites from the Variolink or Multilink product lines, for example.

Monobond Etch & Prime etches and silanizes silicate ceramic surfaces in one working step.

Composition

Alcoholic-aqueous solution of ammonium polyfluoride, silane methacrylate and colourant.

Indication

- Conditioning of silica-based ceramic surfaces for the adhesive bond with luting composites

Contraindication

- Any use not listed in the indication
- If a patient is known to be allergic to any of the ingredients of Monobond Etch & Prime.
- Intraoral application

Side effects

Systemic side effects are not known to date. In rare cases, components of Monobond Etch & Prime may lead to sensitization. The product should no longer be used in such cases.

Interactions

The contamination of surfaces conditioned with Monobond Etch & Prime with saliva and blood must be prevented.

Usage**Conditioning of pretreated and non-pretreated restorations made of silicate ceramic**

(e.g. IPS Empress®, IPS e.max® Press, IPS e.max CAD)

- Following the try-in, thoroughly rinse the restoration with water spray and dry with water- and oil-free air.
- Apply Monobond Etch & Prime on the adhesive surface using a Microbrush and agitate into the surface for 20 seconds using slight pressure. Allow to react for another 40 seconds.
- Thoroughly rinse Monobond Etch & Prime off with water until the green colour has been removed. If residue remains in microporosity after rinsing, the restoration can be cleaned with water in an ultrasonic bath for up to 5 minutes.
- Dry the restoration with a strong stream of water- and oil-free air for approximately 10 seconds.

Seat the conditioned restoration using a luting composite (e.g. Variolink® Esthetic or Multilink® Automix) in accordance with the respective Instructions for Use.

Additional notes

- To avoid any contamination, Monobond Etch & Prime must not be removed directly from the bottle using a microbrush.
- Carefully close the Monobond Etch & Prime bottle immediately after use.

Warnings

- The product contains alcohols. Keep away from sources of ignition.
- Monobond Etch & Prime is corrosive. Avoid contact with the skin and mucous membrane.
If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Do not use Monobond Etch & Prime after the indicated date of expiration.
- Storage temperature 2–28°C (36–82°F)
- Shelf life: see expiry date on label

Keep out of the reach of children!**For use in dentistry only.**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.