Fluor Protector N

- Protective varnish containing fluoride
- Vernis protecteur fluoré
- Barniz protector conteniendo fluoruro

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Instructions for Use

Description

Fluor Protector N is a protective fluoride varnish for tooth desensitization and caries prophylaxis.

Composition

Ethanol, water, film formers, flavourings, 1.5% ammonium fluoride (corresponds to 7700 ppm fluoride).

Indication

- Treatment of hypersensitive tooth necks
- Improvement of enamel resistance against acid
- Long-term caries prophylaxis
- After tooth whitening

Contraindication

If patients are known to be allergic to any of the ingredients in Fluor Protector N, the material should not be used.

Side effects

A slight temporary burning sensation may occur if the product comes in contact with the oral mucous membrane.

Application

Fluor Protector N is applied by dentists and dental professionals.

Note: Children under the age of seven who take systemic fluoride (fluoride tablets) should omit this treatment for several days after the treatment with Fluor Protector N to avoid fluoride overdose.

Generally, Fluor Protector N is applied at six-month intervals. In the event of high caries risk or erosion risk, shorter intervals may be indicated. The contents of the bottle are sufficient for up to 30 full sets of teeth.

Step-by-step procedure

1. Clean the tooth surfaces.
2. Create a dry working field (e.g. with cotton rolls or cotton wool wads).
3. Place the required amount in a dappen dish or similar vessel and close the bottle.
4. Apply a thin varnish layer with the help of an applicator or brush.
5. Let the varnish dry for one minute; thereafter, remove the cotton rolls.
6. After the treatment, ask the patient to spit out the contents of the mouth without rinsing.

Tips for patients

Patients should refrain from eating and drinking for one hour after the treatment with Fluor Protector N.

Storage

- Carefully close the bottle immediately after use.
- The product is highly flammable and should therefore be kept away from ignition sources.
- Do not use the material after the indicated date of expiration.
- Store the product at 2–28 °C (36–82 °F).

Keep the material out of the reach of children! For dental use only!

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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